

GETTING STARTED WITH THE **OBSERV[®] 520_x**

01 DO YOU HAVE A COMPATIBLE IPAD FOR THE
OBSERV 520X APP? (SEE COMPATIBILITY SHEET).

02 WATCH VIDEO ON HOW TO SETUP THE OBSERV
LINKED HERE

03 IPAD VISOR INSTALL: FOR CLEAR IMAGING IT IS
IMPORTANT THAT YOUR CAMERA IS CORRECTLY
PLACED IN THE IPAD VISOR. THE IMAGE
DISPLAYS THE CORRECT WAY OF SETTING UP
YOUR IPAD VISOR

04 CONFIGURE YOUR IPAD:
• CONNECT TO YOUR COMPANY APPLE ID
• SET UP PREFERRED EMAIL ACCOUNT ON
YOUR IPAD VIA:
SETTINGS > MAIL > ACCOUNT > ADD ACCOUNT
THIS EMAIL ACCOUNT WILL BE USED BY THE
OBSERV APP TO SEND OUT TREATMENT
REPORTS TO CLIENTS
• DOWNLOAD OBSERV 520X APP
• REGISTER YOUR SHOP

05 PAIR YOUR IPAD:
• OPEN THE OBSERV APP
• GO TO SETTINGS
• TAP DEVICE OPTIONS
• TAP PAIR WITH NEW DEVICE
• A POP UP MESSAGE "PAIR NEW DEVICE" WILL
APPEAR ON YOUR IPAD SCREEN
• PRESS POWER BUTTON ON OBSERV, IT WILL
PAIR AUTOMATICALLY
• YOU ARE NOW ABLE TO CAPTURE IMAGES

06

REDEEM THE VOUCHER CODES FOR AESTHETIC ANALYSIS AND SKIN DIAGNOSIS

- SETTINGS
- DEVICE OPTIONS
- "REDEEM VOUCHER CODE"
- ENTER THE CODE ON BACK OF THE VOUCHER
- PRESS REDEEM
- PRESS THE POWER BUTTON TO TRANSFER THE PRODUCT TO THE DEVICE

07

CLAIM FREE CLOUD

- GO TO SETTING IN OBSERV 520X APP
- CLICK ON DEVICE OPTIONS
- CLICK ON THE POP-UP CLAIM FREE CLOUD
- FOLLOW THE STEPS AND FILL OUT YOUR COMPANY DETAILS

08

GET CONNECTED WITH SYLTON CONNECT:

- LOGIN VIA ANY BROWSER ON DESKTOP, TABLET WITH SAME CREDENTIALS USED FOR THE OBSERV APP
- HAVE ALL PATIENT DATA AVAILABLE AT ANY PLACE AND TIME
- SAVE PATIENT PHOTOS AND STORE THEM IN THE PATIENT FILES
- EASILY MANAGE PERSONALIZED CATALOGUE BY ADDING TREATMENTS AND PRODUCTS

09

MAKE IT YOUR OWN:

PERSONALIZE YOUR CATALOGUE ON SYLTON CONNECT

- PRESET PRODUCTS FROM BRAND LIBRARY AVAILABLE:
 - ALUMIERMD, BIOLOGIE RECHERCHE, BIOPELLE, COSMEDIX, DERMALOGICA, DMK, IMAGE SKINCARE, IS CLINICAL, JAN MARININ, MEDIK9, MESOESTETIC, MURAD, NEOCUTIS, NEODERMA, NEOSTRATA, OBAGIE, PCA SKIN, QMS, REVISION SKINCARE, SKIN BETTER, SKIN TECH, SKIN CEUTICALS, TOSKANI, ZO

10

CONGRATS!!! YOU HAVE FINISHED GETTING STARTED WITH YOUR OBSERV. NEXT CHECK OUT THE APP MANUAL TO FAMILIARIZE YOURSELF WITH THE OBSERV 520X APP FUNCTION AND FEATURES